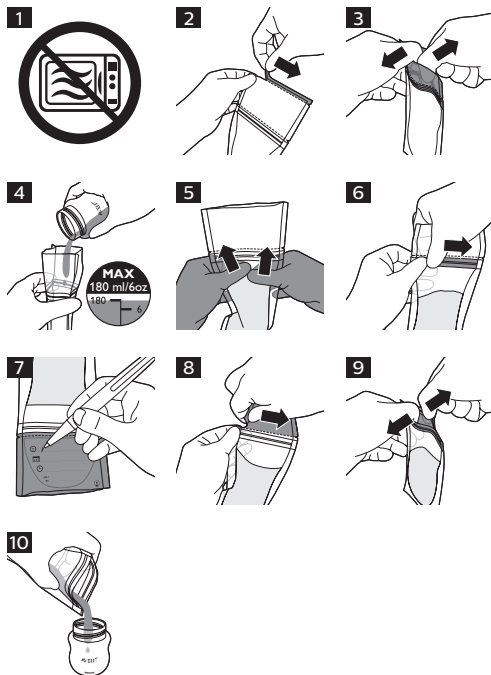
**PHILIPS****AVENT**

Breast Milk  
Storage Bags  
Sachets de  
conservation du lait  
maternel  
أكياس تخزين  
حليب الأم

SCF603

**ENGLISH****Introduction**

The Philips Avent breast milk storage bags offer a safe and convenient way to collect your precious breast milk. They are made of dual-layer, reinforced plastic and have a strong seal, to ensure your milk remains protected while being stored. A wide and sturdy opening ensures safe and easy filling and pouring. For ultimate hygiene, every bag comes disinfected with a tamper-evident seal.

**Important**

Read this user manual carefully before you use the breast milk storage bags and save it for future reference.

**Warning**

- Do not microwave (Fig. 1).
- Keep the breast milk storage bags out of the reach of children.
- Always check the temperature of the milk before you feed your baby.
- To prevent the milk from overheating, do not defrost the breast milk storage bag with breast milk in the microwave, in freshly boiled water or in a bottle warmer.

**Caution**

- The breast milk storage bags are intended for one-time use only.
- Thoroughly wash your hands before you fill and empty the breast milk storage bags.
- The actual amount of milk in the bag may deviate slightly from the level indications on the bag.
- Never refreeze breast milk or add fresh breast milk to already frozen breast milk.
- You can store breast milk in a breast milk storage bag in the refrigerator for up to 4 days or in the freezer for up to 6 months.

**Filling the breast milk storage bag**

- 1 Open the breast milk storage bag by tearing off the tamper-evident seal at the top of the bag. Keep the torn-off seal away from children and dispose of it safely (Fig. 2).
- 2 To open the bag, place your thumbs inside the plastic above the zip and gently pull both sides apart (Fig. 3).
- 3 Pour the expressed milk into the bag. Do not fill the bag beyond the 180 ml/6 oz level, as breast milk expands when frozen (Fig. 4).
- 4 Carefully remove excess air by flattening the part of the bag above the milk (Fig. 5).
- 5 Seal the bag by closing the zip (Fig. 6).
- 6 Use a pen to write down the date of expression and other information on the labeling area. Do not write on the filling part to prevent puncturing the bag (Fig. 7).

**Defrosting breast milk**

To defrost breast milk overnight, place the breast milk storage bag in the refrigerator.

To defrost breast milk prior to feeding, place the breast milk storage bag in warm water.

**Pouring milk from the breast milk storage bag**

- 1 Tear off the labeling area to eliminate potential contamination of the breast milk during pouring (Fig. 8).
- 2 To open the bag, place your thumbs inside the plastic above the zip and pull both sides apart (Fig. 9).
- 3 Hold the bag by the zip and pour the milk into a container (Fig. 10).

Tip: Lightly squeeze the zip to make pouring easier.

- 4 Throw away the breast milk storage bag after use. Do not reuse the breast milk storage bag.
- 5 To heat the milk, place the container with defrosted milk in a bottle warmer or in a bowl of hot water.

Note: Always discard any breast milk that is left over at the end of a feed.

**Support**

If you need information or support, please visit [www.philips.com/support](http://www.philips.com/support).

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Trademarks are the property of Koninklijke Philips N.V.
All rights reserved. / Tous droits réservés.